

Jones Radiology Online

Jones Radiology Online allows referring practitioners access to imaging and reports with the use of IntelViewer (which is a DICOM viewer) or IntelConnect (a no-install, secure web portal).

Getting Started

1. Open IntelViewer from your desktop by double-clicking the icon.
 2. Enter your Username and Password.
 3. Search by using the patient's surname (or part thereof) and first name (e.g. Smith, Mary).
 4. From the search results, double-click the relevant study. IntelViewer will open the images in a new window.
- ① If the relevant study does not appear in the search results, try using the 'Break-Glass' function.

Basic Image Viewing

- | | | | |
|--|---------------|---|------------------|
|  | Drag and drop |  | View port layout |
|  | Zoom |  | Report |
|  | Contrast |  | Search |

Break-Glass


The Break-Glass function is used to access imaging and reports referred by another practitioner.

1. On the right side of the 'Search Tool' panel, click the 'Break-Glass Access' button. (If this button is not available, please contact your Referrer Liaison Officer).
2. Read the terms of the confidentiality agreement and click the acknowledge checkbox.
3. Select the toggle to Break-Glass using Patient Name and DOB.
4. Enter the patient's name (Last name, First name) and date of birth (YYYYMMDD), click 'Search'.
5. Click 'Access Patient Studies'.
6. You will then return to the 'Search Tool' panel where the patient's studies will be displayed.
7. Open the relevant study by double-clicking on it.

IntelConnect

IntelConnect is a no-install, secure, web-portal which can be accessed from any browser, on any device. All you need is your IntelViewer username and password.

<https://online.jonesradiology.com.au/portal>

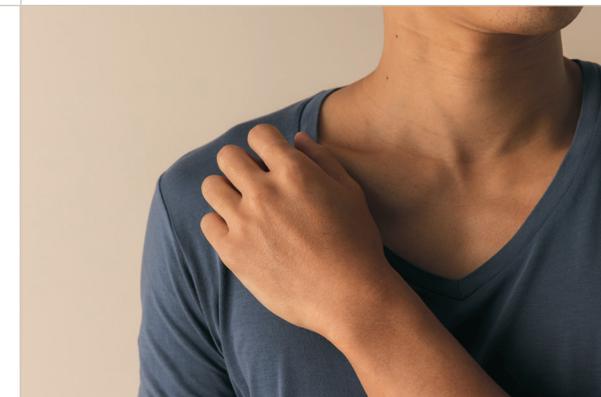

User Access Details

Username:

Password:

Referrer Support

Please visit our website for help and FAQs.

www.jonesradiology.com.au/referrers

For more information or technical assistance, please contact your Referrer Liaison Officer or call 1800 810 655.